

Ramp Assembly Instructions

Thank you for purchasing the Roll-A-Ramp®. You have ordered a length that requires minimal assembly. Please follow these step-by-step instructions to assemble your ramp into its final length.

Step 1: Remove both sections of the ramp from their individual boxes taking special note of “TOP” section and “BOTTOM” section as noted on the boxes.


Step 2: Identify the Owners Manual and tools to assemble your ramp. Also, identify any accessories that you ordered with your ramp. The accessories will be included in a separate cardboard box inside one of the sections of ramp or in the zip-lock bag with your Owners Manual.

Step 3: Remove zip-ties from the ramp sections and then place the ramp on edge and roll out. Place on a flat surface with the top surface of the ramp down.

Step 4: Place two sections close together. One section of the ramp will have a bolt, caution cover and nut attached through the bolt holes. Remove this hardware and move the two sections together so that the tabs are overlapping on both sides and the sides of the ramp are even.



Step 5: Insert bolt with yellow caution cover through both holes. This may take some maneuvering to line up the holes. Put on nut. **If Positive Lock Pins were ordered as an accessory with your ramp, insert them at this step instead of bolts.*

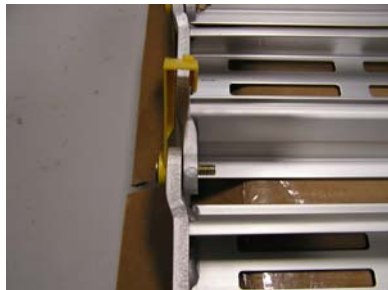

Step 6: Locate box wrench and allen wrench from your Owners Manual packet. Use these tools to tighten the connection on the ramp. Do not over-tighten. Position the caution cover over the pinch point on the side of the ramp.

This concludes the assembly of your Roll-A-Ramp®. Always read your Owners Manual prior to use.