

Front Bag

Part No. B2112

Now with 300 yard visible reflective safety tape!!



Large basketball sized bag that mounts to end of armrest and is easily detachable. Zippered top enclosure for added security. Light weight aluminum frame allows bag to always be open.

Front Bag

Approx. Dimensions 7" Height x 12" Diameter

1 Mounting Receiver: 2 "L" bracket/ straps riveted on the black tube.

1 Bag (with rod attached) First, decide on which side you would like to put the Front Bag. Your Front Bag comes from the factory set for the RIGHT* side (as you sit in the chair.)

The new "L" bracket/strap mounting system allows the Front Bag to mount to most vehicle's armrest without any hardware.

- -Unfasten the 2" webbing strap enough to slide over your armrest (you may need to complete undo the strap if you have any obstruction to sliding on).

 -Keep the "L" bracket to the outside and against the armrest.
- -Feed the strap through the metal buckle, pull to tighten, and fasten the hook and loop.
- -Now you can insert the handle of the Front Bag, locking it in place with the push button. The mounting receiver has a push button setting to lock the bag in an upright position. This makes it easier to get in and out of your vehicle.

*If you are installing your Front Bag on the LEFT side, all you will need to do is rotate the two "L" bracket/straps so that the side of the "L" bracket not riveted to the tube is on the outside of your armrest.

OTHER ARMREST BAGS AVAILABLE FOR YOUR SCOOTER OR WHEELCHAIR ARMREST:

<u>Armrest Pocket:</u> Small enough bag to fit under manual wheelchairs without interfering with back wheel.

<u>Deluxe Saddle Bag:</u> Same size as the Standard Saddle Bag with added side and inside pockets.

Large Saddle Bag: A bag that will not interfere with mid-wheel drive tires.

Extra-Large Saddle Bag: Large storage area with lots of separate storage pockets.

Standard Saddle Bag: Single pocket bag that mounts to any armrest.

We strongly encourage you to call us at (800) 795-2392 if you have any problems, questions, or comments. We are always in development of new products for wheelchair, scooter, crutch, and walker users... So let us know if there's something special you need!

DIESTCO Mfg. Corp. • P.O. Box 6504 • Chico, CA, 95927 • (800) 795-2392 • www.diestco.com