

IMPORTANT INFORMATION

- ▲ Rated load is 850 pounds. **Never exceed rated load.**
- ▲ Read and follow all labels, instructions, safety and maintenance information before use.
- ▲ For additional care, use or safety information, or to obtain replacement copies, please call 1-800-451-1903 or visit www.ezaccess.com.
- ▲ Do not use if ramp is damaged or unstable.
- ▲ For wheelchairs and scooters only (not intended for commercial use).
- ▲ Ramp may be slippery when wet or icy.
- ▲ Use only if sufficient maneuvering room is available at both ends of the ramp to assure safe travel when entering and exiting the ramp.
- ▲ Use ramp only with a qualified helper and always use your lap belt.
- ▲ It is important that you refer to your equipment's (wheelchair or scooter) User Guide for the proper degree of incline or decline and chair direction before attempting ramp use. **Never exceed its recommendations.**

FITTING A MODULAR ENTRY MAT

The mat can be easily trimmed to accommodate lower heights and narrower widths and, if necessary, notched to suit the doorjamb configuration. Alternately, mat can be combined with an additional Modular Entry Mat and one set of Risers to create a maximum height of 4¾".

1. If door threshold is equal to the height of the Modular Entry Mat (2½" high), use mat as is (FIG. 2, section A).
2. If door threshold is less than the height of the Modular Entry Mat (2½" high), trim mat to the desired height, depth and/or width using a reciprocating saw, razor knife, or hand saw (FIG. 1).
 - ▲ Use caution when operating any non-manual saw as serious injury may result due to unexpected binding or pinching of rubber mat material. Always follow tool manufacturer's safety guidelines.
3. If door threshold is greater than the height of the Modular Entry Mat (2½" high), two Modular Entry Mats (shown in FIG. 2, sections A & B) along with one pair of Risers (FIG. 2, section C) can be combined to accommodate a rise up to 4¾" high.

✎ With this configuration, section A must remain at its original height (ok to trim width only).

- a. FIG. 2, section B, can be trimmed to the desired height, depth and/or width. If section B is trimmed, section C, which is used to increase overall height, needs to be trimmed to match the depth and/or width of section B.
- b. Secure section B to section C using supplied Phillips Head Screws.

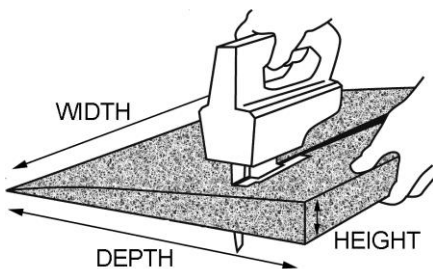


FIG. 1

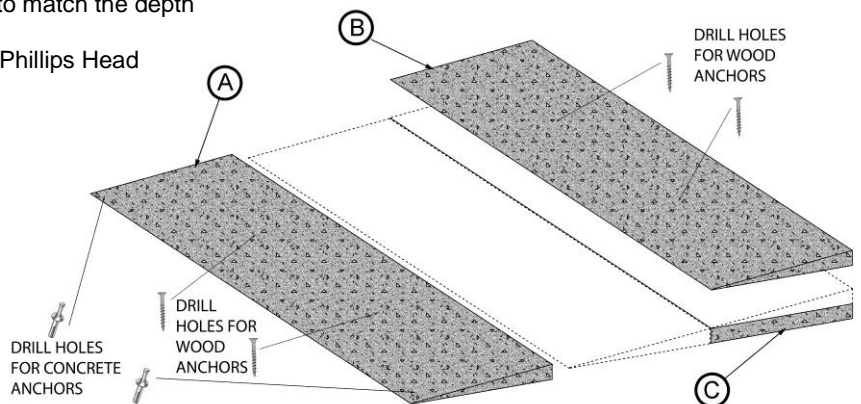


FIG. 2

FASTENING THE MAT

It is recommended that the mat be securely fastened.

1. ATTACHING TO WOOD
 - a. Place mat in desired location and ensure that surface is free of dirt and debris.
 - b. Secure mat using supplied Phillips head screws (2 each). Space screws evenly (see FIG. 2 for placement) and drive until head is ¼" below the mat surface.
2. ATTACHING TO CONCRETE
 - a. Place mat in desired location and ensure that surface is free of dirt and debris.
 - b. Using a ¼" diameter masonry bit, drill a hole 2" deep 1" from the corner edges at the taper (FIG. 2).
 - ✎ Ensure the holes are free of any residue.
 - c. Drive the supplied ¼" diameter mushroom head pin anchors (two each) into the holes with a hammer until the head is seated.

SAFETY AND MAINTENANCE

1. At all times, keep mat clear of snow, ice, dirt, leaves, and other debris that may accumulate on the surface.
2. Clean mat with mild soap and warm water. Rinse well and use extra caution when mat surface is wet.

WARRANTY

Three-year limited warranty. See reverse side of document for details.

FITTING THE MODULAR ENTRY MAT FOR USE AS A CARGO WEDGE

This versatile Modular Entry Mat also adapts to your vehicle. Many vehicles have a 1 – 3" drop from entrance to cargo floor. The Modular Entry Mat can easily be trimmed to the contours of the vehicle's cargo area, filling the void and providing a smooth transition for mobility equipment access.

1. Measure height, width, and length of area to be fitted.
 - ✎ It is recommended that you create a cardboard template of the area before cutting.
2. Place template on top of mat and mark areas (FIG. 3).
3. Using a reciprocating saw, razor knife, or hand saw, trim as needed (FIG. 4).
 - ⚠ Use caution when operating any non-manual saw as serious injury may result due to unexpected binding or pinching of rubber mat material. Always follow tool manufacturer's safety guidelines.
4. Place Modular Entry Mat in recessed area of vehicle (FIG. 5).



FIG. 3

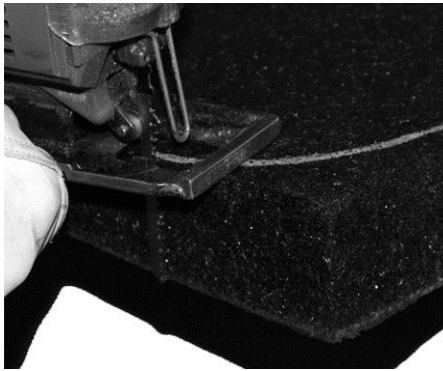


FIG. 4



FIG. 5

3-YEAR LIMITED WARRANTY

1. EZ-ACCESS, a division of Homecare Products, Inc., (herein referred to as Manufacturer), warrants that the equipment shall be free from defects in material and workmanship under normal use and service for a period of 3-years from the date of original purchase. This limited warranty applies to the original purchaser only and is non-transferable. This limited warranty is applicable to defective metal structure components only (excludes powder coated finish, applied non-skid tread, etc.) No part of this warranty will apply to any equipment which has been subject to misuse, vandalism, negligence, alteration, improper loads, accident, improper installation, or which has been repaired outside Manufacturer's place of business in any way as in the reasonable judgment of Manufacturer, to adversely affect its performance and reliability, nor to normal deterioration due to wear, tear and exposure, corrosion, or damage caused by rain, fire, earthquake or other natural causes or acts of nature.
2. Manufacturer's obligation and purchaser's sole remedy under this warranty is limited to, at Manufacturer's option, repairing or replacing equipment which is returned to its place of business and which, upon examination, shall disclose to Manufacturer's reasonable satisfaction to have been defective. Manufacturer will make the repair or replacement of defective components at its own expense. If Purchaser discovers a defect, he/she must call Manufacturer at 253-249-1101 to request a Return Authorization (RA) to return the merchandise to Manufacturer.
3. If Manufacturer determines, in its sole discretion, to replace the equipment with either a new or repaired unit, purchaser shall be responsible for a charge prorated on the following basis: The purchaser will be allowed a credit against the purchase price of equivalent equipment at the time of return to Manufacturer, in proportion to the percentage of the warranted service life remaining at the time of such return. Purchaser will be responsible to pay the balance of the purchase price of equivalent equipment. Manufacturer reserves the right to require payment prior to delivery of the repaired/replaced equipment. In calculating available credit, the remaining warranted service life will be rounded up or down to the nearest whole year. YEARS 0-1: Unit repaired or replaced; YEARS 1-2: Repair or 50% \$ credit (current purchase price multiplied by credit percentage); YEARS 2-3: Repair or 25% \$ credit (current purchase price multiplied by credit percentage). This limited warranty does not cover removal or reinstallation. Manufacturer reserves the right to require Purchaser to pay for all shipping charges (Government purchases exempt).
4. **DISCLAIMER: THE WARRANTIES AND REMEDIES CONTAINED HEREIN ARE EXCLUSIVE AND IN LIEU OF ALL OTHER WARRANTIES, WHETHER EXPRESS, IMPLIED OR STATUTORY, INCLUDING ANY LIABILITY ARISING UNDER ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, STATUTORY OR OTHERWISE. HOWEVER, THAT IF ANY IMPLIED WARRANTY CANNOT BE DISCLAIMED IN ANY TERRITORY WHERE A PRODUCT IS SOLD, THE DURATION OF SUCH IMPLIED WARRANTY IS LIMITED TO THE DURATION OF THE LIMITED WARRANTY DESCRIBED IN THE FIRST PARAGRAPH.**
5. **MANUFACTURER SHALL NOT BE LIABLE UNDER ANY CONTRACT, NEGLIGENCE, STRICT LIABILITY OR OTHER LEGAL OR EQUITABLE THEORY FOR ANY DIRECT, SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, RESULTING FROM THE USE, MISUSE OR INABILITY TO USE THE EQUIPMENT, OR ARISING OUT OF ANY BREACH OF THIS LIMITED WARRANTY, EVEN IF MANUFACTURER HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. THE SOLE REMEDY FOR A BREACH OF THE FOREGOING LIMITED WARRANTY IS REPLACEMENT OF THE DEFECTIVE OR NON-CONFORMING EQUIPMENT OR PART. THE MAXIMUM LIABILITY OF MANUFACTURER UNDER THIS WARRANTY IS LIMITED TO THE PURCHASE PRICE OF THE EQUIPMENT COVERED BY THE WARRANTY.**
6. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you. This warranty gives you specific legal rights, and you may also have other rights which may vary from State to State. Manufacturer reserves the right to change or improve its products without any obligation to change or improve any Product previously manufactured by it.