



Single Fold Aluminum Wheelchair Ramp -600LB Capacity

Instructions for parts#: SCG-2, SCG-3, SCG-4, SCG-5, SCG-6, SCGPLUS-2, SCGPLUS-3, SCGPLUS-4, SCGPLUS-5, SCGPLUS-6

GENERAL GUIDELINES & SAFETY

- It is the user's responsibility to read and follow all instructions
- Keep these instructions with the product at all times and review before each use
- It is the responsibility of this product's owner to furnish the instructions to any person that borrows or purchases the product
- Inspect the product before use to ensure it is assembled properly and all parts are in safe working order and free of defects
- Never modify this product in any way
- All circumstances cannot be addressed in these instructions
- Please use common sense and practice general safety measures when using this product
- Parts and/or instructions are subject to change without notice
- Do not exceed the rated 600 lb. capacity of your ramp
- Do not exceed the maximum height limit for the height of your ramp
- Use caution at all times when loading and unloading
- Load and unload onto a stable, level surface
- NEVER stand in the path of equipment or vehicle. Keep bystanders and children away.
- DO NOT leave attached ramps unattended.
- Beware of hinged areas. Keep all body parts clear of hinged areas.
- Ramp surfaces will have less traction when wet. Use caution.
- The ramp is not a toy. DO NOT use ramp as a jump, scaffold, bridge, ladder or anything other than its designed purpose.
- ALWAYS make sure the top of the ramp attaching lip is secured onto step, landing or vehicle before using. We strongly recommend using the supplied safety pins to prevent ramp kick out.
- ALWAYS follow manufacturer recommendations for chair or scooter.
- Maintain the ramp by regular cleaning. Periodically lubricate the hinges.

MAXIMUM *OCCUPIED / **UNOCCUPIED HEIGHT LIMIT TO GROUND LEVEL

2' ramp: Occupied loading: 4" maximum, unoccupied loading height: 6"

3' ramp: Occupied loading: 6" maximum, unoccupied loading height: 9"

4' ramp: Occupied loading: 8" maximum, unoccupied loading height: 12"

5' ramp: Occupied loading: 10" maximum, unoccupied loading height: 15"

6' ramp: Occupied loading: 12" maximum, unoccupied loading height: 18"

*Occupied is a person sitting in the wheelchair/scooter while loading (2:12 slope or 9.6 degree angle)

**Unoccupied is loading with nobody sitting in the wheelchair/scooter (3:12 slope or 14.5 degree angle)

LOADING WHEELCHAIR

Step 1: Position the wheelchair in the direction of the ramp with front wheel forward. Roll or ride the wheelchair slowly and steadily up the ramp, ensuring that the wheels follow the center of the ramp.

Step 2: Take extra care in wet conditions to ensure the wheels do not slide on the ramp.

UNLOADING WHEELCHAIR

Step 1: Position the ramp as detailed above directly behind the rear wheel of the wheelchair. Allow the wheelchair to roll backwards down the ramp, ensuring the wheels follow the center of the ramp.

Step 2: Always have a qualified assistant present when using the ramp.

PRODUCT WARRANTY & LIABILITY

GENERAL PRODUCT WARRANTY: Products purchased from the Authorized Dealer (original place of purchase) or Merchant ("Dealer") will be free of defects in material and workmanship at the time of receipt, and will meet the specifications stated at the place of purchase transaction or online at the Dealer's website, under normal use and service when correctly installed, operated and maintained. This product warranty is effective for the period of time stated below, unless otherwise stated in the product instructions or depicted in the product advertisement. All Authorized Dealer warranties are NON-TRANSFERABLE and cover only the original end purchaser. This limited warranty does not cover products purchased through non-authorized dealers. Non-authorized dealer receipts are not accepted for warranty verification.

*****WARRANTY CLAIMS MUST BE MADE DIRECTLY TO THE ORIGINAL PLACE OF PURCHASE.*****

WARRANTY PERIOD: This warranty remains in force for one year from the date of the product's accepted delivery. The Authorized Dealer offers a one year manufacturer's warranty for most products unless otherwise specified on the product advertisement.

WARRANTY SERVICE: The Authorized Dealer will replace any defective or malfunctioning part at no charge, including payment of the shipping costs of parts or replacement product to and from the manufacturer. The purchaser is responsible for labor charges. If the product does not meet specifications as depicted in the advertisement, the Authorized Dealer will refund the full purchase price of the product.

Questions regarding the warranty on a specific product and warranty claims should be directed to the Authorized Dealer with whom the purchase transaction was made.

WARRANTY LIMITATIONS: The above warranty does not apply to products that are repaired, modified or altered by any party other than the Authorized Dealer; are subjected to unusual physical stress or conditions (such as overloaded ramps or corrosion), natural disaster, governmental actions, wars, strikes, labor disputes, riots, theft, vandalism, terrorism or any reason beyond reasonable control; are damaged due to improper installation, misuse, abuse, accident or negligence in use, storage, transportation or handling, or tampering; or to products that are considered consumable items or items requiring repair or replacement due to normal wear and tear.

Product should be inspected prior to signing for delivery. Product damage incurred during shipping, unless noted on the Bill of Lading at the time of delivery, renders this warranty void.

LIMITED LIABILITY: In no event shall **THE AUTHORIZED DEALER** be liable to the purchaser or any third party for any indirect, incidental, consequential, special, exemplary or punitive damages arising out of the use of the product, including, without limitation, property damage, loss of value of the product or any third party products that are used in or with the product, or loss of use of the product or any third party products that are used in or with the product.

