


Aluminum Modular Self-Supporting Threshold Ramp

Instructions for Part # THRX Series

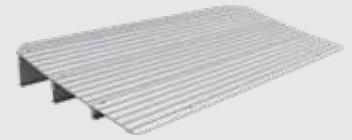

General Guidelines ⚠

- It is the user's responsibility to read and follow all instructions.
- Keep these instructions with the product at all times and review before each use.
- It is the responsibility of this product's owner to furnish the instructions to any person that borrows or purchases the product.
- Inspect the product before use to ensure it is assembled properly and all parts are in safe working order and free of defects.
- Never modify this product in any way.
- All circumstances cannot be addressed in these instructions. Please use common sense and practice general safety measures when using this product.
- Parts and/or instructions are subject to change without notice.


Safety ⚠

- This ramp has a weight capacity of 600 lbs. Do NOT exceed this weight.
- The ramp is available in several different sizes and it is the user's responsibility to measure the height of the entryway and choose the right size.
- Refer to the user guide for the wheelchair, scooter, etc. for proper degree of incline. Do NOT exceed its recommendation.
- Stop using ramp if it becomes damaged or unstable.

Parts

Part	Description	Quantity
A	Self-tapping screw	1-18
B	Flat head bolt	4
C	Adjustable top flap	1
D	Support foot	0-5
E	Ramp extension	0-5
F	Bottom section	1

*Quantities and size may vary depending on size of ramp

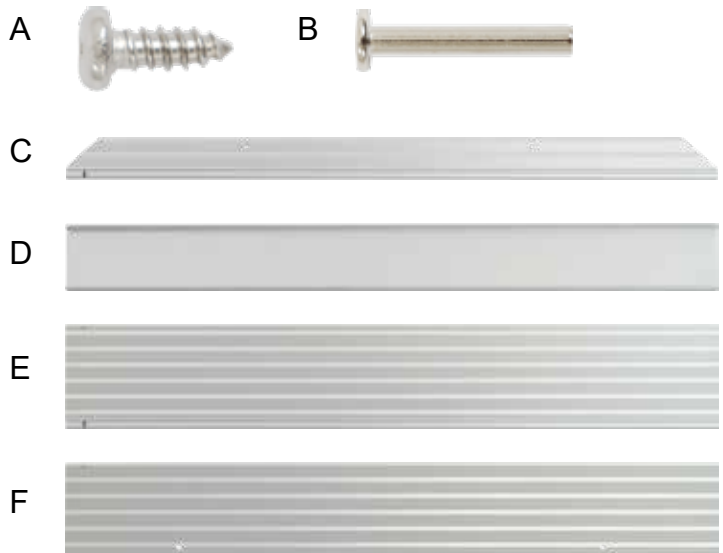

Assembly

Note: When assembling ramps THR1 and THR15, please start with Step 3 in the assembly instructions for ramps THR2, THR3, THR4, THR5, and THR6, please start with Step 1.

Step 1. Align the opening on the bottom of a ramp extension with the top of the ramp's support foot. (If the purchased ramp has multiple ramp extension pieces, please note that the extension pieces are all the same size.)

Slide the support foot into the opening on the bottom of the ramp extension.

Fig. 1a



Fig. 1b



Align both holes in the support foot with both holes in the ramp extension. Once in place, use two self-tapping screws to secure in place.


Repeat with additional support feet (if necessary).

Fig. 1c



Step 2. Once all support feet are secured to the bottom of the ramp extensions, organize sections from lowest to highest height. Each ramp extension will have to be secured to the piece next to it.

Fig. 2a

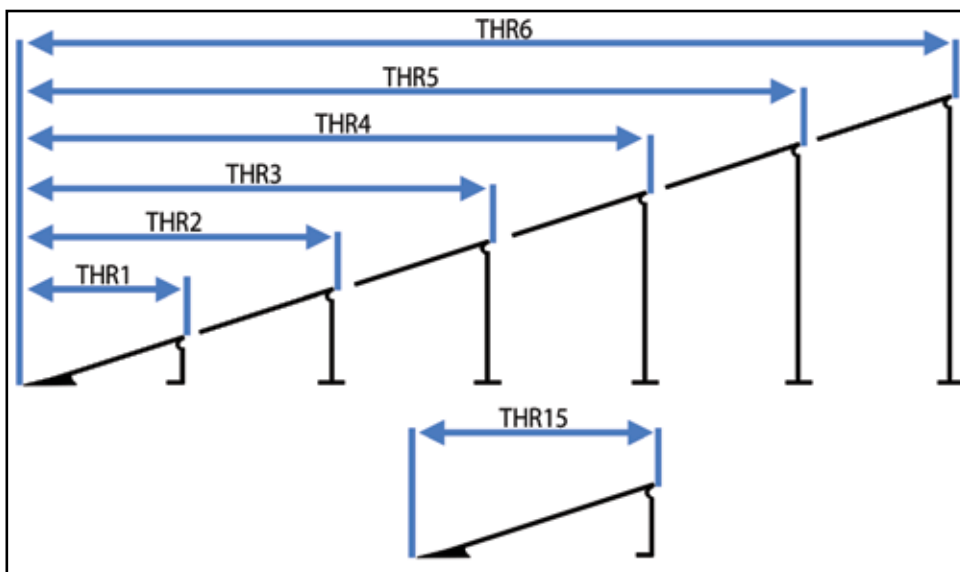

(Step 2 Continued) After it's determined where each piece will go, align the front of the ramp extension into the back of another extension piece and slide together until the screw holes match up.

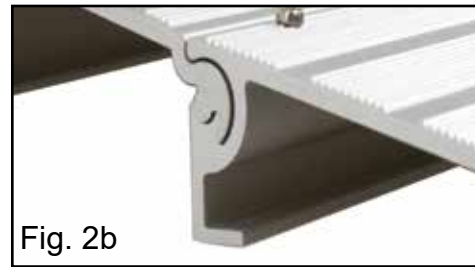


Fig. 2b

Use one tapping screw to secure each ramp section together. Repeat until all extension pieces are secured to each other.



Fig. 2c

Step 3. The adjustable top flap is optional during the use of the ramp. The top flap should NOT be used when it may interfere with opening and closing a door.

Align the end of the adjustable flap to the end of the highest ramp extension piece. Slide the flap through until the screw holes align.

Use one tapping screw to secure the flap to the ramp.



Fig. 3

Installation

Step 1. Once assembled, take the ramp to the desired location and put the back flush against an elevated threshold.

Step 2. For permanent mounting, mark the surface below where the two holes are on the adjustable flap (optional) and the two holes are on the base of the ramp. Drill holes where marked.

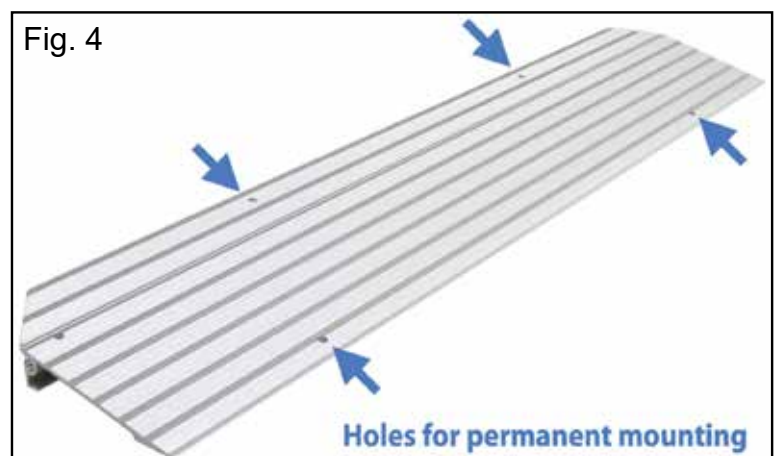


Fig. 4

Step 3. Use included flat head bolts for securing the ramp into place through the mounting holes shown above.

Product Warranty and Liability

GENERAL PRODUCT WARRANTY: Products purchased from the Authorized Dealer (original place of purchase) or Merchant (“Dealer”) will be free of defects in material and workmanship at the time of receipt, and will meet the specifications stated at the place of purchase transaction or online at the Dealer’s website, under normal use and service when correctly installed, operated and maintained. This product warranty is effective for the period of time stated below, unless otherwise stated in the product instructions or depicted in the product advertisement. **All Authorized Dealer warranties are NON-TRANSFERABLE** and cover only the original end purchaser. This limited warranty does not cover products purchased through non-authorized dealers. Non-authorized dealer receipts are not accepted for warranty verification.

*****Warranty claims must be made directly to the original place of purchase.*****

WARRANTY PERIOD: This warranty remains in force for one year from the date of the product’s accepted delivery. The Authorized Dealer offers a one year manufacturer’s warranty for most products unless otherwise specified on the product advertisement.

WARRANTY SERVICE: The Authorized Dealer will replace any defective or malfunctioning part at no charge, including payment of the shipping costs of parts or replacement product to and from the manufacturer. The purchaser is responsible for labor charges. If the product does not meet specifications as depicted in the advertisement, the Authorized Dealer will refund the full purchase price of the product.

Questions regarding the warranty on a specific product and warranty claims should be directed to the Authorized Dealer with whom the purchase transaction was made.

WARRANTY LIMITATIONS: The above warranty does not apply to products that are repaired, modified or altered by any party other than the Authorized Dealer; are subjected to unusual physical stress or conditions (such as overloaded ramps or corrosion), natural disaster, governmental actions, wars, strikes, labor disputes, riots, theft, vandalism, terrorism or any reason beyond reasonable control; are damaged due to improper installation, misuse, abuse, accident or negligence in use, storage, transportation or handling, or tampering; or to products that are considered consumable items or items requiring repair or replacement due to normal wear and tear.

Product should be inspected prior to signing for delivery. Product damage incurred during shipping, unless noted on the Bill of Lading at the time of delivery, renders this warranty void.

LIMITED LIABILITY: In no event shall THE AUTHORIZED DEALER be liable to the purchaser or any third party for any indirect, incidental, consequential, special, exemplary or punitive damages arising out of the use of the product, including, without limitation, property damage, loss of value of the product or any third party products that are used in or with the product, or loss of use of the product or any third party products that are used in or with the product.

