



# Ramp Wheel Kit

## Instructions for Part # WC-WHEEL-KIT



### General Guidelines

- It is the user's responsibility to read and follow all instructions.
- Keep these instructions with the product at all times and review before each use.
- It is the responsibility of this product's owner to furnish the instructions to any person that borrows or purchases the product.
- Inspect the product before use to ensure it is assembled properly and all parts are in safe working order and free of defects.
- Never modify this product in any way.
- All circumstances cannot be addressed in these instructions. Please use common sense and practice general safety measures when using this product.
- Parts and/or instructions are subject to change without notice.

### Safety

- The ramp being moved must have an approach lip where the underside is accessible when folded.
- The axle will fit ramps, when folded, up to 18" wide.
- Always make sure the ramp is secured with the axle straps and handle strap before moving.
- When securing the handle strap, be sure the buckle is resting on the inside of the ramp edge to prevent slipping.

### Parts



Part	Description	Quantity
A	Handle Strap	1
B	Axle Strap Male End	1
C	Axle Strap Female End	1

Part	Description	Quantity
D	Axle	1
E	Wheel	2
F	Axle End Cap	2

Tools Required: Hammer

# Assembly

**Step 1.** Fit one wheel (E) onto the end of the axle (D). Place one end cap (F) on the end of the axle where the wheel was placed, then hammer into place. ***\*Do not attach the second wheel until after Step 2.***



**Step 2.** Slide the two axle straps (B, C) onto the axle (D).



**Step 3.** Attach the second wheel and end cap as described in Step 1.



# Use

**Handle Strap:** Secure the handle strap (A) around the foot-end of the ramp to be moved. Make sure the strap buckle is resting to the inside of the ramp edge to prevent slipping. Tighten securely.



**Wheeled Axle:** Place the wheeled axle underneath the ramp lip, so the axle fits into the corner created by the edge of the ramp and the ramp lip. Secure the ramp to the axle using the attached axle straps, and tighten securely.



# Product Warranty and Liability

**GENERAL PRODUCT WARRANTY:** Products purchased from the Authorized Dealer (original place of purchase) or Merchant (“Dealer”) will be free of defects in material and workmanship at the time of receipt, and will meet the specifications stated at the place of purchase transaction or online at the Dealer’s website, under normal use and service when correctly installed, operated and maintained. This product warranty is effective for the period of time stated below, unless otherwise stated in the product instructions or depicted in the product advertisement. **All Authorized Dealer warranties are NON-TRANSFERABLE** and cover only the original end purchaser. This limited warranty does not cover products purchased through non-authorized dealers. Non-authorized dealer receipts are not accepted for warranty verification.

**\*\*\*Warranty claims must be made directly to the original place of purchase.\*\*\***

**WARRANTY PERIOD:** This warranty remains in force for one year from the date of the product’s accepted delivery. The Authorized Dealer offers a one year manufacturer’s warranty for most products unless otherwise specified on the product advertisement.

**WARRANTY SERVICE:** The Authorized Dealer will replace any defective or malfunctioning part at no charge, including payment of the shipping costs of parts or replacement product to and from the manufacturer. The purchaser is responsible for labor charges. If the product does not meet specifications as depicted in the advertisement, the Authorized Dealer will refund the full purchase price of the product.

Questions regarding the warranty on a specific product and warranty claims should be directed to the Authorized Dealer with whom the purchase transaction was made.

**WARRANTY LIMITATIONS:** The above warranty does not apply to products that are repaired, modified or altered by any party other than the Authorized Dealer; are subjected to unusual physical stress or conditions (such as overloaded ramps or corrosion), natural disaster, governmental actions, wars, strikes, labor disputes, riots, theft, vandalism, terrorism or any reason beyond reasonable control; are damaged due to improper installation, misuse, abuse, accident or negligence in use, storage, transportation or handling, or tampering; or to products that are considered consumable items or items requiring repair or replacement due to normal wear and tear.

Product should be inspected prior to signing for delivery. Product damage incurred during shipping, unless noted on the Bill of Lading at the time of delivery, renders this warranty void.

**LIMITED LIABILITY:** In no event shall THE AUTHORIZED DEALER be liable to the purchaser or any third party for any indirect, incidental, consequential, special, exemplary or punitive damages arising out of the use of the product, including, without limitation, property damage, loss of value of the product or any third party products that are used in or with the product, or loss of use of the product or any third party products that are used in or with the product.

